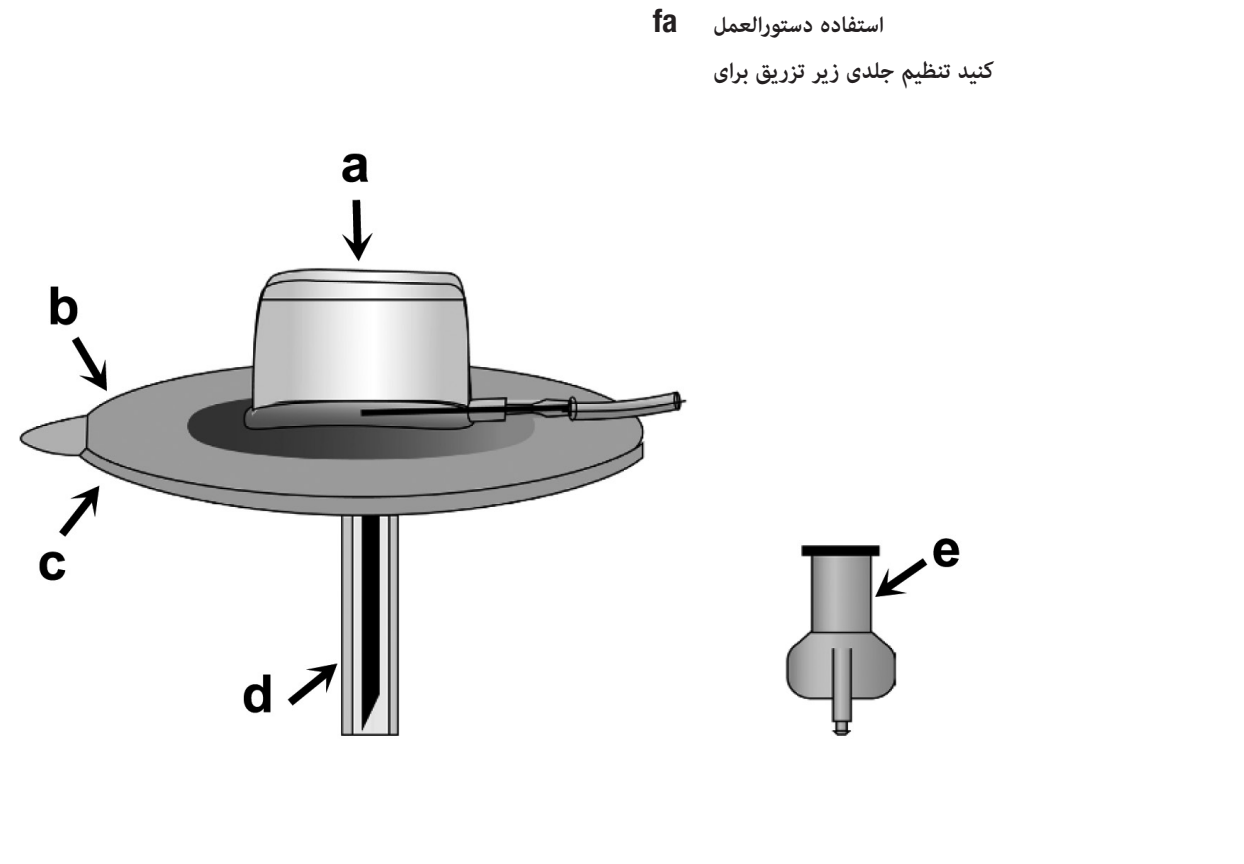
D-mine[®] Infset EVER PHARMA

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en The D-mine[®] Infset is a subcutaneous cannula.

The *D-mine[®]* Infset is equipped with a removable gripping pin (a) and non-irritant adhesive (b) which is covered with a backing film (c) on the underside. There is a protective cap covering the needle(d).

Warnings

The *D-mine[®]* Infset is only intended for subcutaneous use and it may not be used for aspiration of venous access ports! It is recommended that the patient be closely monitored for development of

compartment syndrome. If insulin is administered, a blood sugar test must be performed 1 to 2 hours after start of infusion.

Instructions

- Please refer to the package leaflet of the medication and your physician’s instructions for administration of your infusion treatment.
- Take the *D-mine[®]* Infset out of the packaging, twist off the protective cap at the end of the tubing and attach the LuerLock connector (e) of the *D-mine[®]* Infset to the pump LuerLock attachment.
- Fill the *D-mine[®]* Infset with the solution to be administered (for priming volume, see Table).
- Prepare skin disinfection.
- Remove the backing film (c) and the protective cap on the needle (d), taking care not to touch it.
- Now hold the *D-mine[®]* Infset between the thumb and forefinger by the gripping pin (a) and with the thumb and forefinger of the other hand pinch a skin fold.
- Insert the needle under the skin and press the adhesive film (b) hard onto the skin. The non-irritant adhesive ensures the *D-mine[®]* Infset remains safely in place.
- Near the puncture site, make two to three strain loops for the catheter system and attach these to the skin with medical tape or a plaster.

Nota

The length of time the *D-mine[®]* Infset may be in place depends on the individual and hospital policy. We recommend to change the *D-mine[®]* Infset after 24 hours. The puncture site should be checked regularly and changed immediately should there be any reddening of the skin or tissue hardening. *D-mine[®]* Infset is a single use item. Should a knik occur in the tubing, the position of the *D-mine[®]* Infset must be adjusted to allow proper drug administration. Should the gripping pin (a) proof difficult to manage after attachment to the skin, it can be removed.

Remove the D-mine[®] Infset

To remove the *D-mine[®]* Infset, remove the adhesive film (b) starting at the edges from the skin and pull it, together with the needle, vertically upwards. Check that the cannula is still on the infusion set. For safe disposal of the used infusion set use a sharps bin.

Dimensions and Priming Volumes D-mine[®] Infset

| | |
|---------------|------------------|
| Tubing length | 80 cm |
| Quantity | 0,12 ml = 120 µl |
| Article-No. | 64750 |

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